HUMANTECHNIK

earis®  
MAX

**Ladestativ**  
**Charging stand**  
**Support de chargement**  
**Oplaadstand**  
**Soporte de carga**

**DE**

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Seite 2

**EN**

**User manual**

Page 4

**FR**

**Mode d'emploi**

Page 6

**NL**

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Pagina 8

**ES**

**Manual de instrucciones**

Página 10

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## **Inductive charging stand for EarisMAX headphones**

With this charging stand, your EarisMAX headphones can be charged efficiently and conveniently without needing to connect a cable.

### **Package contents**

- earisMAX charging stand
- USB-A to USB-C cable
- USB power supply with country adapter
- User manual
- Safety instructions
- Warranty card

### **Setup**

Insert the USB-C plug of the connection cable into the connection on the rear edge of the stand base. Insert the USB-A plug into the connection on the power supply unit.

Connect the power supply unit to a domestic power socket. The charging indicator light on the charging stand will briefly light up purple.

### **Charging the EarisMAX headphones**

Now hang the EarisMAX headphones on the bracket of the charging stand. The charging indicator lights up blue after a few seconds and the headphone battery is charging.

When the battery is fully charged, the charging indicator lights up green for 1 minute and then goes out. The headphones can now be removed from the charging stand and used.

The charging stand only needs approx. 150 minutes to fully charge a completely depleted battery.

**Note:** The inductive charging function is only compatible with EarisMAX headphones. Products from other brands cannot be charged using this function.

### **Specifications:**

Dimensions (without headphones): Ø 140 mm x H 232 mm

Weight (without headphones): 180 g

Input voltage: 5 V

Output power: 1.5 W.

Operating principle: electromagnetic induction, operating frequency 160KHz

Operating temperature: -10°C to ~45°C

Storage temperature: -40°C to ~70°C

Specifications are subject to change without prior notice.

This device fulfils the requirements of the following EU directives:

- 2011/ 65/EU + 2015/863/EU RoHS Directive

- 2012/19/EU WEEE Directive

- 2014/53/EU RED Directive

Conformity with the above directives is confirmed by the CE mark on the device. CE declarations of conformity are available on the Internet at [www.humantechnik.com](http://www.humantechnik.com).

Humantechnik's UK authorised representative:

Sarabec Ltd, 15 High Force Road, MIDDLESBROUGH TS2 1RH

United Kingdom

Sarabec Ltd., hereby declares that this device complies with all UK statutory instruments. UK declaration of conformity available from: Sarabec Ltd.

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